AUDYZEN®

DAILY CLEAN

MEDICAL DEVICE CLASS IIa

Audyzen® Daily Clean is a 100% natural and preservatives free ear spray based on isotonic sea water and purified water for daily ear hygiene. It allows cleaning effectively the auditory canal, ensuring its correct state and avoiding wax build-up that can cause problems such as hearing loss.

Ingredients:

Purified water and sea water (sodium chloride 0.9%).

Instructions of use and recommended dose:

- Shake the bottle before use. The first time it is used, it may require pressing several times the applicator until the pulverization of the product.
- Gently insert the applicator in the ear and apply the product. For adults and children from 1 year, apply 1-2 sprays per day.
- After each application, clean the applicator with water and dry.
- In case the applicator is removed from the spray bottle, insert the applicator and press until you hear the click.

Contraindications and adverse effects:

- No adverse effects have been described at the recommended therapeutic doses, but if you experience side effects, consult your usual physician or pharmacist.
- Any serious adverse event related to this product must be declared to the manufacturer, distributor and national health authority in your country.
- At the indicated doses, no interactions are known. Similarly, it is recommended to consult with your physician if you are taking any medication.

Warnings and precautions:

- Do not use in case of any physiological or pathological discomfort in the ear.
- In case of pain or discomfort in the ear, do not use the product and consult a health care professional.
- For hygienic reasons and to prevent the transmission of pathogens, this container should not be used by more than one person.
- Do not put any element in the ear before or after using **Audyzen® Daily Clean**.
- Keep out of the reach of children.
- For proper use of the product, it is recommended to keep box and leaflet while using it.

Conservation and expiry date:

- Do not use the product after the expiry date shown on the box and label.
- Store in a cool and dry place.
- Discard one month after opening.
- Deposit the bottle and the ear applicator in the corresponding containers.

Presentation

30 ml spray without propellant gas in a bottle with ear applicator.

100 ml spray without propellant gas in a bottle with ear applicator, available in soft force.

Date of issue: 17/07/2024. Version 02.

Knowing your experience with the product
will allow us to grow

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