FREE NOSE® PROPOLIS WITH THYME & EUCALYPTUS - SALINE SOLUTION - NASAL SPRAY

FREE NOSE® PROPOLIS WITH THYME & EUCALYPTUS is a medical device for adults and children from 12 years, formulated with a saline solution with 1% of sodium chloride. It is indicated in the nostrils hygiene by dragging in pathological processes as rhinitis, excessive secretions, flu and catarrhal processes.

It also contains extracts of propolis, thyme and eucalyptus, which provide it a pleasant and fresh aroma.

Composition:

Purified water, sodium chloride (1%), propolis glycerinated extract, EDTA disodium salt dehydrate, thyme glycerinated extract, benzalkonium chloride, eucalyptus fluid extract and sodium hydroxide.

Directions of use:

1. Shake the bottle before use and remove the protective cap. The first time it is used, it may require pressing several times the applicator until the pulverization of the product.
2. Introduce gently the nasal applicator in one nostril and press the pump to apply the product with a light massage on the area.
3. Let drain the excess solution in order to extract the mucus, and then blow.
4. Repeat the process with the other nostril.
5. Clean the tip with water and dry.
6. Cover the nasal applicator with the protective cap until the next use.

Recommended dose:

For adults and children from 12 years: 1 or 2 sprays, 2 to 3 times per day in each nostril if necessary.

Contraindications and adverse effects:

- No adverse effects have been described at the recommended therapeutic doses, but if you experience side effects, consult your usual physician or pharmacist.
- First treatment can produce aqueous rhinorrhea banal, natural due to the effect of the components.
- Any serious adverse event related to this product must be declared to the manufacturer, distributor and national health authority in your country.
- At the indicated doses, no interactions are known. Similarly, it is recommended to consult with your physician if you are taking any medication.

Warnings and precautions:

- For hygienic reasons and to prevent the transmission of pathogens, this container should not be used by more than one person.
- In case of known or suspected allergy to any of the components of this medical device, as a precaution, it is advisable not to use.
- Keep out of reach of children.

Conservation and expiry date:

- Do not use the product after the expiry date shown on the box and the bottle.
- Maintain in a dry and cool place. Keep below 40°C.
- Discard 3 months after opening.
- Deposit the bottle and the protective cap in the corresponding containers.
- It is recommended not to discard the box or the leaflet during the use of the product.

Presentation:

Spray without propellant gas in a 20 ml bottle with nasal applicator.

Date of issue: 17-07-2024. Version 03.

Knowing your experience with the product will allow us to grow

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