FREE NOSE® HYALURONIC PLUS - NASAL SPRAY

FREE NOSE® HYALURONIC PLUS is a nasal spray featuring with Hyaluronic acid and Dexpanthenol (pro-vitamin B5) in a physiological saline solution with xanthan gum. This medical device forms a protective film over the mucosa helping restore the natural barrier of the mucous membrane thanks to its content in xanthan gum. Due to its content in in hyaluronic acid and dexpanthenol with moisturizing properties, it is ideal for the prevention and treatment of nasal mucosa dryness. This isotonic saline solution is suitable for adults and children from 1 year of age for prevention of symptoms and as additional therapy in the rhinitis treatment. It is recommended in situations that require the moisturizing of nasal mucosa as consequence of various causes such as rhinitis from flu, allergies, and dry nostrils after surgery, air conditioning and drug treatments.

Composition:

Purified water, dexpanthenol (3%) (pro-vitamin B5), sodium chloride (0.9%), potassium dihydrogen phosphate, dipotassium hydrogen phosphate, xanthan gum (0.3%), sodium hyaluronate (0.2%), EDTA disodium salt dehydrate, benzalkonium chloride and sodium hydroxide.

Directions of use:

1. Shake the bottle before use and remove the protective cap. The first time it is used, it may require pressing several times the applicator until the pulverization of the product.
2. Introduce gently the nasal applicator in one nostril and press the pump to apply the product.
3. Repeat the process with the other nostril.
4. Clean the tip with water and dry.
5. Cover the nasal applicator with the protective cap until the next use.

If significant discomfort appears during its use, stop the treatment and consult a physician.

Recommended dose:

For adults and children from 1 year: apply in each nostril as often as necessary.

Contraindications and adverse effects:

- No adverse effects have been described at the recommended therapeutic doses, but if you experience side effects, consult your usual physician or pharmacist.
- Any serious adverse event related to this product must be declared to the manufacturer, distributor and national health authority in your country.
- At the indicated doses, no interactions are known. Similarly, it is recommended to consult with your physician if you are taking any medication.

Warnings and precautions:

- Keep out of the reach of children.
- In case of a known or suspected allergy to any of the contents of this medical device, as a precaution, it is advisable to avoid using it.
- For hygienic reasons and to prevent the transmission of pathogens, this container should not be used by more than one person.
- The source of hyaluronic acid is not of animal origin. It is obtained from bacterial fermentation.

Conservation and expiry date:

- Do not use after the expiry date printed on the box and bottle.
- Discard 3 months after first opening.
- Keep in a cool and dry place. Keep below 40°C.
- Deposit the bottle and the protective cap in the corresponding containers.
- It is recommended not to discard the box or the leaflet during the use of the product.

Presentation:

Spray without propellant gas in a 20 ml bottle with nasal applicator.

Date of issue: 17-07-2024. Version 04.

Knowing your experience with the product will
allow us to grow

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