FREE NOSE® DEEP BREATH - NASAL SPRAY

Nasal decongestant designed to aid during pathological processes occurring with nasal secretion. The excess mucus in the nasal cavities may increase the microbial proliferation or the excessive dryness of the nasal mucosa. The hypertonic saline solution has a strong moisturizing effect without causing a rebound effect or addiction. The essential oils of Eucalyptus and balsam fir provide a pleasant refreshing aroma that helps to prolong the effect of relief and freshness.

FREE NOSE® DEEP BREATH is a nasal spray for adults and children of 6 years of age and above which can be used in case of infections of the mucous membrane of the nasopharynx, runny nose, infection of the paranasal sinuses and snoring caused by dryness of the membranes. On the basis of the osmotic effect of the hypertonic solution this nasal spray removes excess fluid inside the nasal mucosa and mucous membrane cells. In addition to the mechanical cleaning and decongestion of respiratory tract this osmotic effect helps to reduce the swelling of mucous.

Composition:

Purified water, sodium chloride (2.5%), 1,2-propylene glycol, ethoxylated hydrogenated castor oil, aloe vera, *Eucalyptus radiata* essential oil, balsam fir essential oil, benzyl alcohol, EDTA disodium salt dehydrate, mixture of natural tocopherols, trisodium citrate dihydrate and benzalkonium chloride.

Instructions for use:

1. Shake the bottle before use and remove the protective cap. The first time it is used, it may require pressing several times the applicator until the pulverization of the product.
 2. Introduce gently the nasal applicator in one nostril and press the pump to apply the product.
 3. Repeat the process with the other nostril.
 4. Clean the tip with water and dry.
 5. Cover the nasal applicator with the protective cap until the next use.
- If significant discomfort appears during its use, stop the treatment and consult a physician.

Recommended dose:

Adults and children from 6 years: 1 spray in each nostril up to 3 times a day.

Contraindications and adverse effects:

- No adverse effects have been described at the recommended therapeutic doses, but if you experience side effects, consult your physician or pharmacist.
- Any serious adverse event related to this product must be declared to the manufacturer, distributor and national health authority in your country.
- At the indicated doses, no interactions are known. Similarly, it is recommended to consult with your habitual doctor if you are taking any medication.

Warnings and precautions:

- Keep out of the reach of children.
- It is not recommended during pregnancy or while breastfeeding.
- In case of a known or suspected allergy to any of the contents of this medical device, as a precaution, it is advisable to avoid using it. Contains d-limonene, geraniol and citral, which are naturally present in the balsam fir and eucalyptus radiata essential oils.
- For hygienic reasons and to prevent the transmission of pathogens, this container should not be used by more than one person.

Conservations and expiry date:

- Do not use after the expiry date printed on the box and bottle.
- Discard 3 months after first opening.
- Keep in a cool and dry place. Keep below 40°C.
- Deposit the bottle and the protective cap in the corresponding containers.
- It is recommended not to discard the box or the leaflet during the use of the product.

Presentation:

Spray without propellant gas in a 20 ml bottle with nasal applicator.

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**Knowing your experience with
the product will allow us to grow**

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