AUDYZEN®

EARWAX

MEDICAL DEVICE CLASS IIa

Audyzen® Earwax is an ear spray with docusate sodium, propylene glycol, glycerin and lemon aroma. It is indicated to soften and dissolve the accumulation of earwax and facilitate its extraction, thanks to its content in docusate sodium, a surfactant agent which dissociates the elements that make up the earwax. At the same time it cleans and lubricates the external auditory canal, helping to prevent the earwax accumulation with the aid of propylene glycol and glycerin. The lemon aroma provides a pleasant freshness.

Ingredients:

Glycerin, purified water, propylene glycol, benzyl alcohol, dioctyl sodium sulfosuccinate (0,5%), lemon aroma, EDTA disodium salt dihydrate, benzalkonium chloride and simethicone.

Instructions of use and recommended dose:

- Shake the bottle before use. The first time it is used, it may require pressing several times the applicator until the pulverization of the product.
- Apply the product in the ear to be treated and leave it for 20 minutes. For adults and children from 2 years, apply 2 sprays per day until discomfort disappears.
- Repeat if it is necessary.
- After each application, clean the applicator with water and dry.
- In case the applicator is removed from the spray bottle, insert the applicator and press until you hear the click.

Contraindications and adverse effects:

- No adverse effects have been described at the recommended therapeutic doses, but if you experience side effects, consult your usual physician or pharmacist.
- Any serious adverse event related to this product must be declared to the manufacturer, distributor and national health authority in your country.
- At the indicated doses, no interactions are known. Similarly, it is recommended to consult with your physician if you are taking any medication.

Warnings and precautions:

- Keep out of reach children.
- In case of a known or suspected allergy to any of the ingredients in this product, as a precaution, it is advisable not to use it.
- For hygienic reasons and to prevent the transmission of pathogens, this container should not be used by more than one person.
- In case of pain or discomfort in the ear, do not use the product and consult a health care professional.
- For proper use of the product, it is recommended to keep box and leaflet while using it.

Conservation and expiry date:

- Do not use the product after the expiry date shown on the box and label.
- Store in a cool and dry place.
- Discard the product 3 months after opening.
- Deposit the bottle and the ear applicator in the corresponding containers.

Presentation:

15 ml spray without propellant gas in a bottle with ear applicator.

Date of issue: 17/07/2024. Version 02.

Knowing your experience with the product
will allow us to grow

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