Audyzen[®] Itch Relief is an ear spray with hyaluronic acid, Tea Tree essential oil and extracts of Calendula and Verbena in alcoholic base. The product relieves the discomfort of the external auditory canal, lubricating and protecting it thanks to its components, thus providing relief to the discomfort caused.

Ideal to relieve discomfort caused by the continued use of hearing aids, headphones and other devices that with the prolonged use can damage the skin of the external ear canal.

Audyzen[®] Itch Relief is a good adjuvant of treatments for ear skin conditions such as eczema, dermatitis and sensitization of the skin of the ear.

Ingredients:

Purified water, ethanol, Calendula and Verbena fluid extract, ethoxylated hydrogenated castor oil, sodium hyaluronate (0.2%), Tea Tree essential oil, EDTA disodium salt dihydrate and benzalkonium chloride.

Instructions of use and recommended dose:

- Shake the bottle before use. The first time it is used, it may require pressing several times the applicator until the pulverization of the product.
- Gently insert the applicator in the external part of the ears and apply the product. For adults and children from 6 years, use as often as necessary.
- After each application, clean the applicator with water and dry.
- In case the applicator is removed from the spray bottle, insert the applicator and press until you hear the click.

The device is aimed for self-administration. Continued use of the product for more than 30 days is not recommended. There is no need for other healthcare professionals assistance.

Contraindications and adverse effects:

- No adverse effects have been described at the recommended therapeutic doses, but if you experience side effects, consult your usual physician or pharmacist.
- Any serious adverse event related to this product must be declared to the manufacturer, distributor and national health authority in your country.
- At the indicated doses, no interactions are known. Similarly, it is recommended to consult with your physician if you are taking any medication.
- It is not recommended to use this product during pregnancy or while breastfeeding

Warnings and precautions:

- Keep out of reach children.
- In case of a known or suspected allergy to any of the ingredients in this product, as a precaution, it is advisable not to use it.
- For hygienic reasons and to prevent the transmission of pathogens, this container should not be used by more than one person.
- In case of discomfort, pain or other ear diagnosed conditions, do not use the product and consult a health care professional.
- For proper use of the product, it is recommended to keep box and leaflet while using it.
- For elderly users, consult the suitability of using the device with your usual doctor or pharmacist.
- In case that discomfort persists after 30 days, do not use the product and consult a health care professional.

Conservation and expiry date:

- Do not use the product after the expiry date shown on the box and label.
- Store in a cool and dry place.
- Discard the product 3 months after opening.
- Deposit the bottle and the ear applicator in the corresponding containers.

Presentation:

20 ml spray without propellant gas in a bottle with ear applicator.

Date of issue: 28-11-2024. Version 02.

**Knowing your experience with the product
will allow us to grow**

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