## FREE NOSE® VIRPROTECT - NASAL SPRAY

**FREE NOSE® Virprotect.** is a decongestant nasal spray based on hypertonic saline solution, sesame oil, glycerol, xanthan gum and natural essences such as tea tree and lemon essential oils. Thanks to its content in xanthan gum, it forms a protective film over the mucosa preventing the contact of allergens (pollen, mites, animal hair, germs...). In addition, due to the acid pH of the solution (3-4), it helps to inactivate microorganisms and the hypertonic solution contributes to their elimination through the osmotic effect.

### Composition:

Purified water, polysorbate 80, sesame oil, glycerol, sodium chloride 2.2%, benzyl alcohol, xanthan gum 0.4%, tea tree essential oil, citric acid, potassium sorbate, EDTA disodium salt dihydrate, benzalkonium chloride, lemon essential oil and mix of natural tocopherols.

### Directions of use:

1. Shake the bottle before use and remove the protective cap. The first time it is used, it may require pressing several times the applicator until the pulverization of the product.
  2. Introduce gently the nasal applicator in one nostril and press the pump to apply the product.
  3. Repeat the process with the other nostril.
  4. Clean the tip with water and dry.
  5. Cover the nasal applicator with the protective cap until the next use.
- The device is aimed for self-administration. There is no need for other health care professionals assistance.

### Recommended dose:

For adults and children from 12 years of age, apply 1 to 3 sprays in each nostril per day.  
Continued use of the product for more than 30 days is not recommended.

### Contraindications and adverse effects:

- No adverse effects have been described at the recommended therapeutic doses of **FREE NOSE® Virprotect**, but if you experience side effects, consult your physician or pharmacist.
- Any serious adverse event related to this product must be declared to the manufacturer, distributor and national health authority in your country.
- At the indicated doses, no interactions are known. Similarly, it is recommended to consult with your physician if you are taking any medication.

### Warnings and precautions:

- In case of a known or suspected allergy to any of the contents of this medical device, as a precaution, it is advisable to avoid using it.
- For hygienic reasons and to prevent the transmission of pathogens, this container should not be used by more than one person.
- Keep out of reach of children.
- For elderly users, consult the suitability of using the device with your usual doctor or pharmacist.
- It is not recommended during pregnancy or while breastfeeding.
- In case of discomfort, pain or other nasal conditions, do not use the product and consult a health care professional.
- In case that discomfort persists after 30 days, do not use the product and consult a health care professional.

### Conservation and expiry date:

- Keep in a cool and dry place. Maintain below 40°C and protected from direct sunlight or other sources of heat.
- Do not use after the expiry date printed on the box and bottle.
- Discard after 3 months once opened.
- Do not discard the content of this device down the drain. Deposit the bottle and the protective cap in the corresponding containers.
- It is recommended not to discard the box or the leaflet during the use of the product.

### Presentation:

- Spray without propellant gas in a 20 ml bottle with nasal applicator.

Date of issue: 07/03/2025.Version 01.

Knowing your experience with the product will allow us to grow

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